


LIGHTGUARD SIZING GUIDE INSTRUCTIONS

This simple to use sizing guide will help you select the LightGuard OverRx size that most accurately fits over your prescription eyewear. To use the guide, set your print preferences with no scaling and print the document at 100% actual size. Then place your eyewear face down on the images of each size. If your eyewear fits within the blue box, the size can be worn over your glasses. If your eyewear exceeds the box, then you must try the next size up. The ideal fit will come from the size that is just slightly larger than your prescription eyewear.


LIGHTGUARD MEDIUM (M)

Designed to be worn over Rx frames not exceeding 136 mm wide x 44 mm high ($5\frac{3}{8}'' \times 1\frac{3}{4}''$)

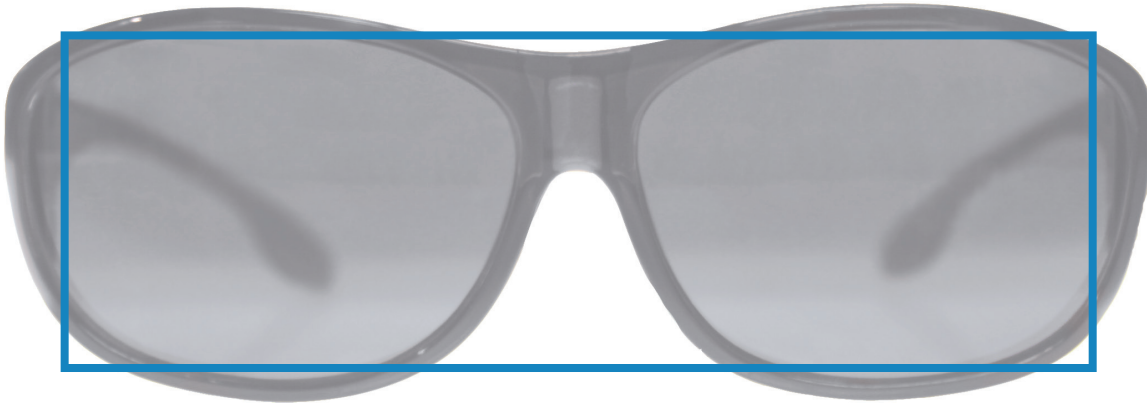

LIGHTGUARD LARGE (L)

Designed to be worn over Rx frames not exceeding 140 mm wide x 50 mm high ($5\frac{1}{2}'' \times 2''$)

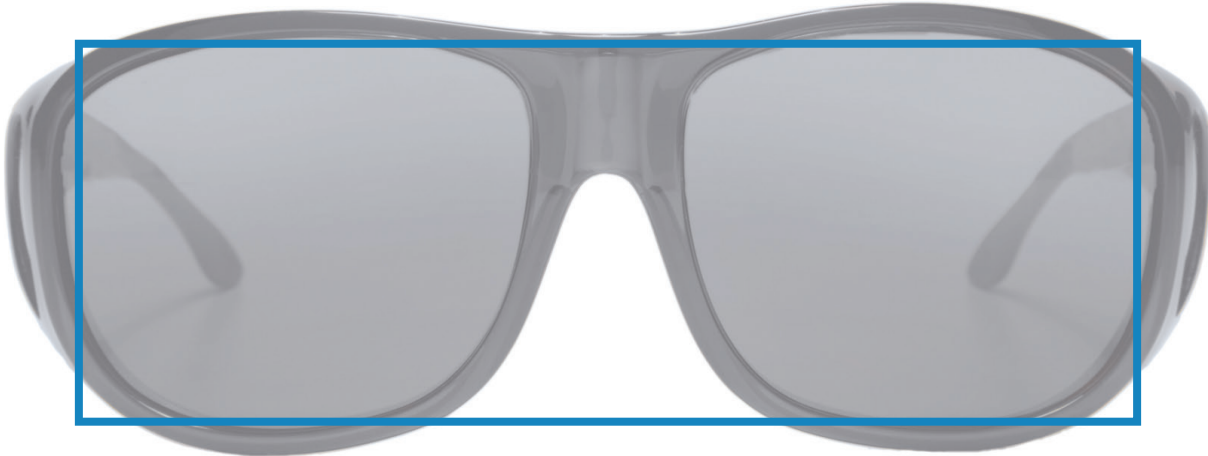

VERIFY THE DOCUMENT PRINTED CORRECTLY

When this document is printed at 100% size, this box should be the same dimensions as a standard credit card.

If the box is smaller or larger than your credit card, you'll need to reprint the document and double check that your printer settings are set to print the page at 100% size (no scaling).

LIGHTGUARD®